



• For in vitro diagnostic use
• Store at 34~86°F (1~30°C) sealed

Test

Humasis HCV Tests Provide You More Reliable and Accurate Test Results.

- **Fast & Easy**
Result in 15 minutes, or less
Simple to use
- **Whole Blood Test**
Whole blood or serum/plasma can be used
- **Highly immunogenic viral proteins are used**
Core, NS3, NS4 and NS5 recombinant proteins are used
- **Convenient**
Room temperature storage
18 months shelf life



HUMASIS Co., Ltd.

Innovative Diagnostics for Improving People's Lives...

One Step HCV Whole Blood Test



Test

For professional use

Test Procedure

1. Cassette

Whole blood/Serum/Plasma

10~15min



Sample collection



20μL

Dispensing



Add 80~100μL
(3~4drops) of
assay diluent



Interpretation of results

2. Multi-device

Whole blood/Serum/Plasma

10~15min



Sample collection



10μL

Dispensing



Add 80~100μL
(3~4drops) of
assay diluent



Interpretation of results



Positive



Negative



Invalid

Specifications

Product name	Humasis HCV Card	Humasis HCV Card(Multi)
Cat. No.	ACB-7030	ACB-6100
Format	Cassette	Multi-device
Specimen	Whole blood or Serum / Plasma	
Storage	34~86°F (1~30°C)	
Shelf life	18 months	
Package	30 Tests/Case	10 × 10 Tests/Case

HUMASIS Co., Ltd.

Innovative Diagnostics for Improving People's Lives...

345-1, DANGJEONG-DONG, GUNPO-SI,
GYEONGGI-DO 435-832, KOREA
TEL: +82-31-459-1524, 1525 FAX: +82-31-459-1526